


UltOS

Plating System

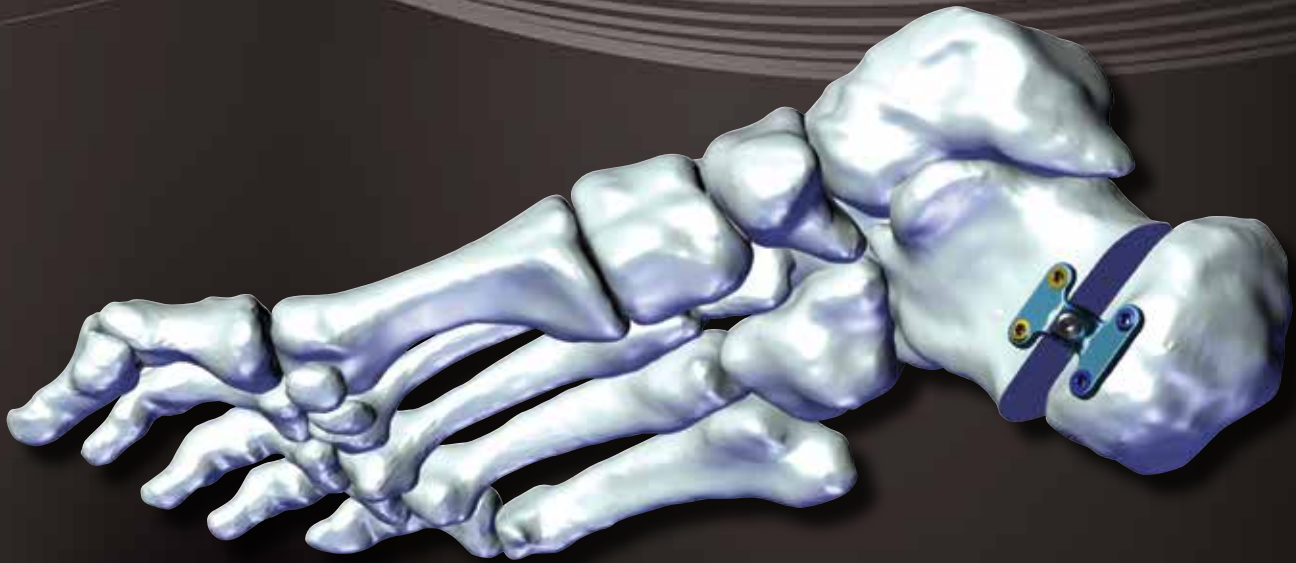

Patient-focused
Surgeon-driven

/ Introduction

The Ortho Solutions UltOS Foot Plating System has been designed to address a number of indications forefoot, midfoot and rearfoot reconstructive surgery. The system achieves an efficient and stable outcome to help minimise intraoperative complications and postoperative morbidity.


Indications

The *intended use* of the Ortho Solutions UltOS Plating System fixation device(s) is to draw two or more aligned small bone fragments together to facilitate healing.


The Ortho Solutions UltOS Plating System is indicated for use in stabilization and fixation of fresh fractures, revision procedures, joint fusion and reconstruction of small bones in the feet, ankles and toes skeleton.

This implant should only be used with the Ortho Solutions UltOS Plating System. Combination with other implants or instrumentation is not permissible.

The Ortho Solutions UltOS Plating System is not intended for spinal use.

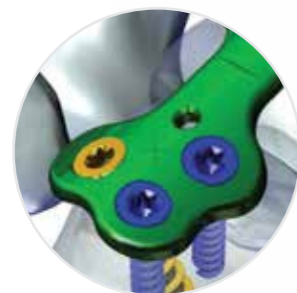

All plates within the system are compatible with both the 2.7mm and 3.5mm screw

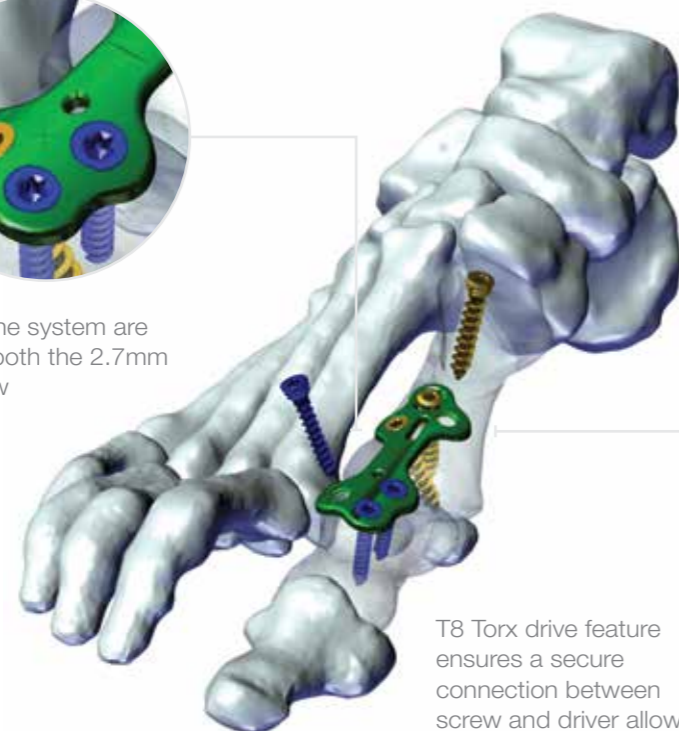

Titanium Alloy Screws and Plates (Ti-6AL-4V ELI)

Divergent screw axis improving plate fixation and construct stability




T8 Torx drive feature ensures a secure connection between screw and driver allowing increased torque transmission

Both locking and non-locking screws sit Sub-flush to plate ensuring a low overall height and minimising soft tissue impingement

/ Compression & Distraction

The Ortho Solutions UltOS Foot Plating System compression and distraction device has been designed for use in a variety of forefoot procedures. "PARALLEL" compression and distraction can be achieved with this ergonomic device. The device can be locked in position and fine adjustments made using the free-wheeling thumb screws. The jaws have guides for both 1.6mm and 2.0mm wires.

Parallel movement of guides in compression and distraction

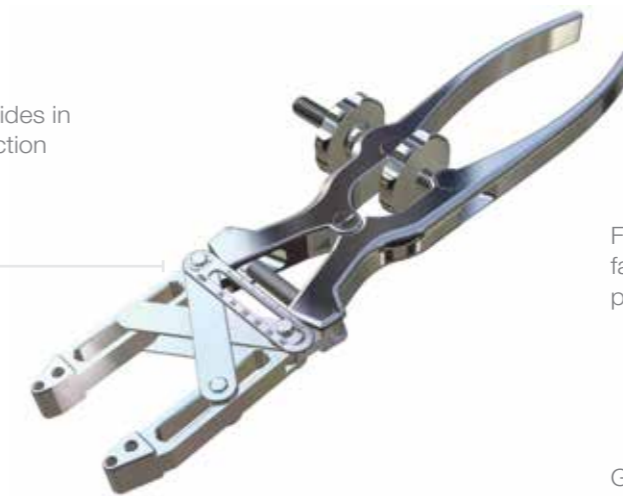

Free-wheeling thumb screws for fast, accurate adjustment and positional locking



Graduated scale for accurate measurement of achieved compression / distraction




/ Construct

The screw holes in the Ortho Solutions UltOS plate range are designed to accept both the 2.7mm and 3.5mm locking and non-locking screws. This will allow for the most appropriate screw diameter choice based on procedure, anatomy, patient bone quality and clinical outcome. The Ortho Solutions UltOS system includes a 3.5mm cannulated compression screw.



All plates are compatible with 2.7mm and 3.5mm locking and non-locking screws

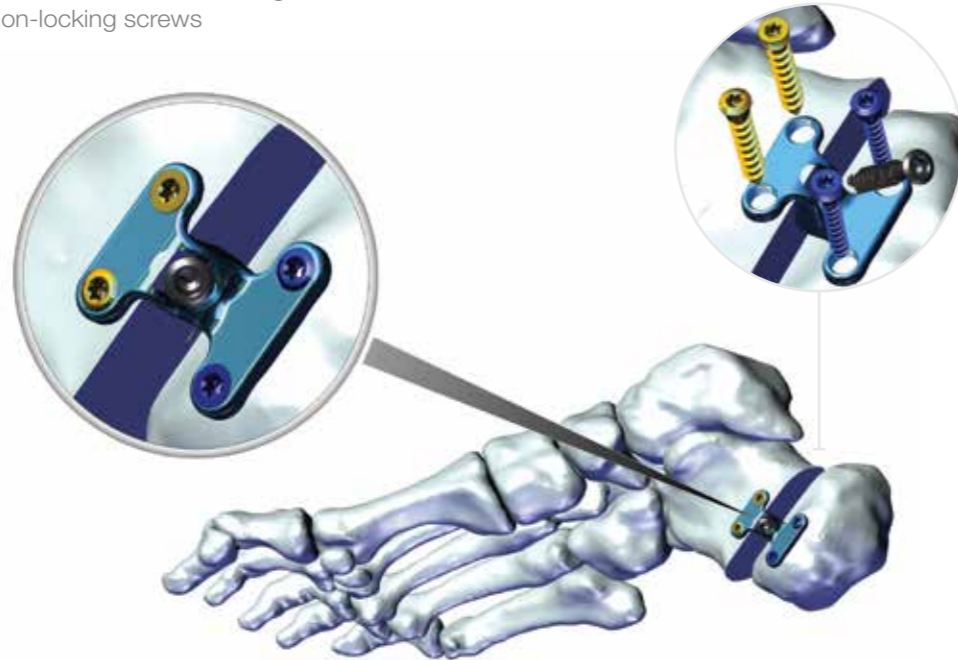

All plate and screw combinations are possible. Plates are compatible with 2.7mm and 3.5mm locking screws. Where the plate has a dynamic compression screw position the 3.5mm cannulated compression screw is available

/ Screws

The UltOS plating system has been designed around offering the Ultimate in construct possibilities. The available locking and non-locking screw options available in both 2.7mm and 3.5mm all have a uniform head design meaning plates are compatible with all variations ensuring the most appropriate screw diameter choice based on procedure, anatomy, patient bone quality and clinical outcome.

The allowable interchange with the non-locking head format allows for controlled "off-axis" alignment.



OS422608-30 Non-Locking 2.7mm



OS422508-30 Locking 2.7mm



OS421814-50 Non-Locking 3.5mm




OS421710-50 Locking 3.5mm



OS326925-60 3.5mm Cannulated Screw Self Drill/Tap

AWP

/ Arthrodesis Wedge Plate

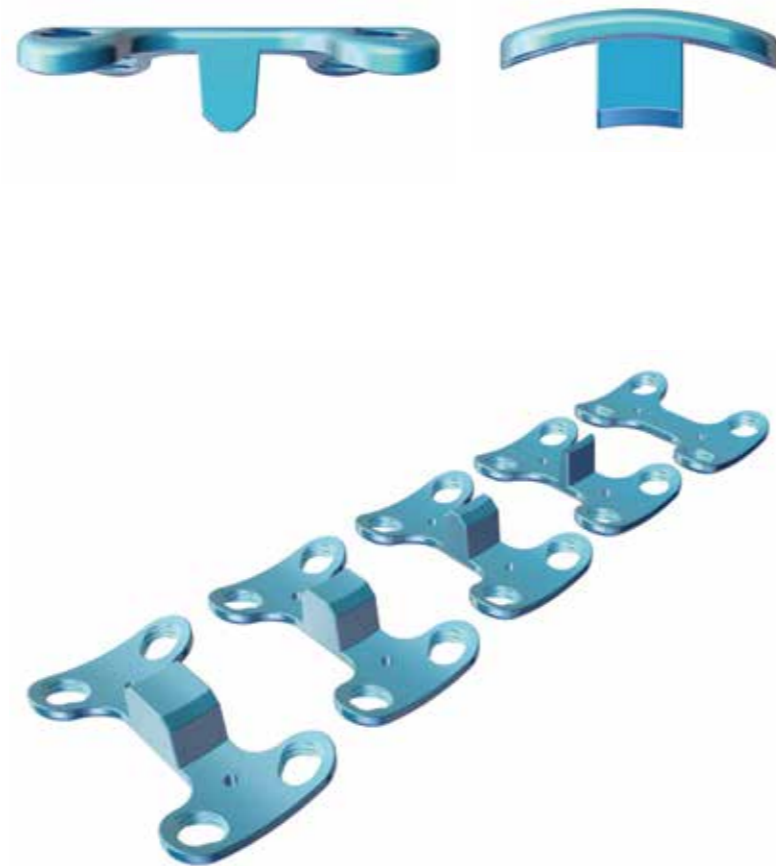

PART NUMBER	DESCRIPTION
OS421300	Arthrodesis Wedge Plate 0mm
OS421302	Arthrodesis Wedge Plate 2mm
OS421304	Arthrodesis Wedge Plate 4mm
OS421306	Arthrodesis Wedge Plate 6mm
OS421308	Arthrodesis Wedge Plate 8mm

/ General Fusion Straight



PART NUMBER	DESCRIPTION
OS422702	General Fusion Straight Plate 2 hole
OS422703	General Fusion Straight Plate 4 hole

GFS

GFT

MTP

/ General Fusion "T"



PART NUMBER	DESCRIPTION
OS422712	General Fusion "T" Plate 2 hole
OS422714	General Fusion "T" Plate 4 hole
OS422716	General Fusion "T" Plate 6 hole

/ Metatarsophalangeal Plate



PART NUMBER	DESCRIPTION
OS42260S-L (or R)	MTP Plate Small Left & Right
OS42260M-L (or R)	MTP Plate Medium Left & Right
OS42260L-L (or R)	MTP Plate Large Left & Right

OWL

/ Opening Wedge Locking Plate



PART NUMBER	DESCRIPTION
OS422400	Opening Wedge Locking Plate 0mm
OS422403	Opening Wedge Locking Plate 3mm
OS422404	Opening Wedge Locking Plate 4mm
OS422405	Opening Wedge Locking Plate 5mm
OS422406	Opening Wedge Locking Plate 6mm
OS422407	Opening Wedge Locking plate 7mm

/ Opening Wedge Plate



PART NUMBER	DESCRIPTION
OS422800	Opening Wedge Plate 0mm
OS422825	Opening Wedge Plate 2.5mm
OS422803	Opening Wedge Plate 3mm
OS422804	Opening Wedge Plate 4mm
OS422805	Opening Wedge Plate 5mm
OS422807	Opening Wedge Plate 7mm

OWP

/ Calcaneal Step Plate



PART NUMBER	DESCRIPTION
OS421608	Calcaneal Step Plate 8mm
OS421610	Calcaneal Step Plate 10mm
OS421612	Calcaneal Step Plate 12mm

/ General Fusion "X" Plates



PART NUMBER	DESCRIPTION
OS42150XS	UltOS General Fusion X Plate XS
OS42150S	UltOS General Fusion X Plate S
OS42150M	UltOS General Fusion X Plate M
OS42150L	UltOS General Fusion X Plate L

LAP


RRP

/ Lapidus Plates



PART NUMBER	DESCRIPTION
OS421000L or R	Lapidus Plate 0mm Left & Right
OS421002L or R	Lapidus Plate 2mm Left & Right
OS421004L or R	Lapidus Plate 4mm Left & Right
OS421006L or R	Lapidus Plate 6mm Left & Right

/ Rearfoot Reconstruction Plates

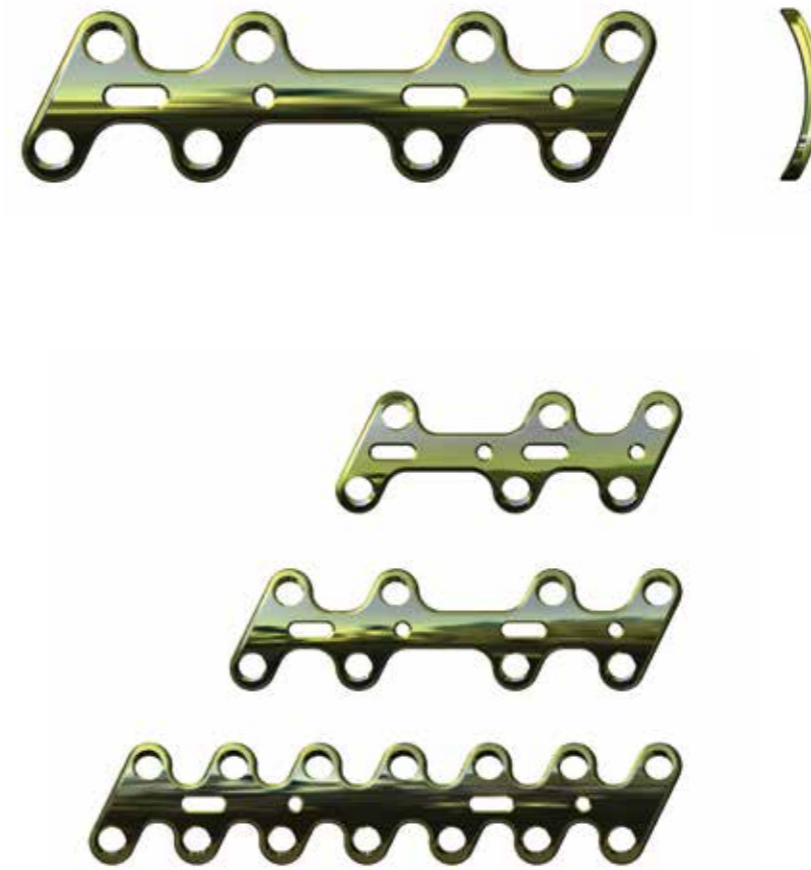

PART NUMBER	DESCRIPTION
OS421406	Rear-foot Recon Plate 6 Hole
OS421408	Rear-foot Recon Plate 8 Hole
OS421414	Rear-foot Recon Plate 14 Hole

TFP

/ Tarsal Fusion Plates



PART NUMBER	DESCRIPTION
OS421512	Tarsal Fusion Plate 12mm
OS421514	Tarsal Fusion Plate 14mm
OS421516	Tarsal Fusion Plate 16mm

/ Universal Locking Plates



PART NUMBER	DESCRIPTION
OS421112	Universal Locking Plate 12mm
OS421116	Universal Locking Plate 16mm
OS421120	Universal Locking Plate 20mm
OS421124	Universal Locking Plate 24mm
OS421130	Universal Locking Plate 30mm

ULP

/ Product Listing

UltOS Plating System - IMPLANTS

OS PART NO.	PART DESCRIPTION	QTY./PACK	STERILITY
OS422608	UltOS - 2.7MM NON-LOCKING SCREW 8MM	2	Sterile
OS422610	UltOS - 2.7MM NON-LOCKING SCREW 10MM	2	Sterile
OS422612	UltOS - 2.7MM NON-LOCKING SCREW 12MM	2	Sterile
OS422614	UltOS - 2.7MM NON-LOCKING SCREW 14MM	2	Sterile
OS422616	UltOS - 2.7MM NON-LOCKING SCREW 16MM	2	Sterile
OS422618	UltOS - 2.7MM NON-LOCKING SCREW 18MM	2	Sterile
OS422620	UltOS - 2.7MM NON-LOCKING SCREW 20MM	2	Sterile
OS422622	UltOS - 2.7MM NON-LOCKING SCREW 22MM	2	Sterile
OS422624	UltOS - 2.7MM NON-LOCKING SCREW 24MM	2	Sterile
OS422626	UltOS - 2.7MM NON-LOCKING SCREW 26MM	2	Sterile
OS422628	UltOS - 2.7MM NON-LOCKING SCREW 28MM	2	Sterile
OS422630	UltOS - 2.7MM NON-LOCKING SCREW 30MM	2	Sterile
OS422608-NS	UltOS - 2.7MM NON-LOCKING SCREW 8MM - IN TRAY	8	Non-Sterile
OS422610-NS	UltOS - 2.7MM NON-LOCKING SCREW 10MM - IN TRAY	8	Non-Sterile
OS422612-NS	UltOS - 2.7MM NON-LOCKING SCREW 12MM - IN TRAY	8	Non-Sterile
OS422614-NS	UltOS - 2.7MM NON-LOCKING SCREW 14MM - IN TRAY	8	Non-Sterile
OS422616-NS	UltOS - 2.7MM NON-LOCKING SCREW 16MM - IN TRAY	8	Non-Sterile
OS422618-NS	UltOS - 2.7MM NON-LOCKING SCREW 18MM - IN TRAY	8	Non-Sterile
OS422620-NS	UltOS - 2.7MM NON-LOCKING SCREW 20MM - IN TRAY	8	Non-Sterile
OS422622-NS	UltOS - 2.7MM NON-LOCKING SCREW 22MM - IN TRAY	8	Non-Sterile
OS422624-NS	UltOS - 2.7MM NON-LOCKING SCREW 24MM - IN TRAY	8	Non-Sterile
OS422626-NS	UltOS - 2.7MM NON-LOCKING SCREW 26MM - IN TRAY	8	Non-Sterile
OS422628-NS	UltOS - 2.7MM NON-LOCKING SCREW 28MM - IN TRAY	8	Non-Sterile
OS422630-NS	UltOS - 2.7MM NON-LOCKING SCREW 30MM - IN TRAY	8	Non-Sterile

OS PART NO.	PART DESCRIPTION	QTY./PACK	STERILITY
OS422508	UltOS - 2.7MM LOCKING SCREW 8MM	2	Sterile
OS422510	UltOS - 2.7MM LOCKING SCREW 10MM	2	Sterile
OS422512	UltOS - 2.7MM LOCKING SCREW 12MM	2	Sterile
OS422514	UltOS - 2.7MM LOCKING SCREW 14MM	2	Sterile
OS422516	UltOS - 2.7MM LOCKING SCREW 16MM	2	Sterile
OS422518	UltOS - 2.7MM LOCKING SCREW 18MM	2	Sterile
OS422520	UltOS - 2.7MM LOCKING SCREW 20MM	2	Sterile
OS422522	UltOS - 2.7MM LOCKING SCREW 22MM	2	Sterile
OS422524	UltOS - 2.7MM LOCKING SCREW 24MM	2	Sterile
OS422526	UltOS - 2.7MM LOCKING SCREW 26MM	2	Sterile
OS422528	UltOS - 2.7MM LOCKING SCREW 28MM	2	Sterile
OS422530	UltOS - 2.7MM LOCKING SCREW 30MM	2	Sterile
OS422508-NS	UltOS - 2.7MM LOCKING SCREW 8MM - IN TRAY	8	Non-Sterile
OS422510-NS	UltOS - 2.7MM LOCKING SCREW 10MM - IN TRAY	8	Non-Sterile
OS422512-NS	UltOS - 2.7MM LOCKING SCREW 12MM - IN TRAY	8	Non-Sterile
OS422514-NS	UltOS - 2.7MM LOCKING SCREW 14MM - IN TRAY	8	Non-Sterile
OS422516-NS	UltOS - 2.7MM LOCKING SCREW 16MM - IN TRAY	8	Non-Sterile
OS422518-NS	UltOS - 2.7MM LOCKING SCREW 18MM - IN TRAY	8	Non-Sterile
OS422520-NS	UltOS - 2.7MM LOCKING SCREW 20MM - IN TRAY	8	Non-Sterile
OS422522-NS	UltOS - 2.7MM LOCKING SCREW 22MM - IN TRAY	8	Non-Sterile
OS422524-NS	UltOS - 2.7MM LOCKING SCREW 24MM - IN TRAY	8	Non-Sterile
OS422526-NS	UltOS - 2.7MM LOCKING SCREW 26MM - IN TRAY	8	Non-Sterile
OS422528-NS	UltOS - 2.7MM LOCKING SCREW 28MM - IN TRAY	8	Non-Sterile
OS422530-NS	UltOS - 2.7MM LOCKING SCREW 30MM - IN TRAY	8	Non-Sterile

OS PART NO.	PART DESCRIPTION	QTY./PACK	STERILITY
OS421814	UItOS - 3.5MM NON-LOCKING SCREW 14MM	2	Sterile
OS421816	UItOS - 3.5MM NON-LOCKING SCREW 16MM	2	Sterile
OS421818	UItOS - 3.5MM NON-LOCKING SCREW 18MM	2	Sterile
OS421820	UItOS - 3.5MM NON-LOCKING SCREW 20MM	2	Sterile
OS421822	UItOS - 3.5MM NON-LOCKING SCREW 22MM	2	Sterile
OS421824	UItOS - 3.5MM NON-LOCKING SCREW 24MM	2	Sterile
OS421826	UItOS - 3.5MM NON-LOCKING SCREW 26MM	2	Sterile
OS421828	UItOS - 3.5MM NON-LOCKING SCREW 28MM	2	Sterile
OS421830	UItOS - 3.5MM NON-LOCKING SCREW 30MM	2	Sterile
OS421835	UItOS - 3.5MM NON-LOCKING SCREW 35MM	2	Sterile
OS421840	UItOS - 3.5MM NON-LOCKING SCREW 40MM	2	Sterile
OS421845	UItOS - 3.5MM NON-LOCKING SCREW 45MM	2	Sterile
OS421850	UItOS - 3.5MM NON-LOCKING SCREW 50MM	2	Sterile
OS421814-NS	UItOS - 3.5MM NON-LOCKING SCREW 14MM - IN TRAY	8	Non-Sterile
OS421816-NS	UItOS - 3.5MM NON-LOCKING SCREW 16MM - IN TRAY	8	Non-Sterile
OS421818-NS	UItOS - 3.5MM NON-LOCKING SCREW 18MM - IN TRAY	8	Non-Sterile
OS421820-NS	UItOS - 3.5MM NON-LOCKING SCREW 20MM - IN TRAY	8	Non-Sterile
OS421822-NS	UItOS - 3.5MM NON-LOCKING SCREW 22MM - IN TRAY	8	Non-Sterile
OS421824-NS	UItOS - 3.5MM NON-LOCKING SCREW 24MM - IN TRAY	8	Non-Sterile
OS421826-NS	UItOS - 3.5MM NON-LOCKING SCREW 26MM - IN TRAY	8	Non-Sterile
OS421828-NS	UItOS - 3.5MM NON-LOCKING SCREW 28MM - IN TRAY	8	Non-Sterile
OS421830-NS	UItOS - 3.5MM NON-LOCKING SCREW 30MM - IN TRAY	8	Non-Sterile
OS421835-NS	UItOS - 3.5MM NON-LOCKING SCREW 35MM - IN TRAY	8	Non-Sterile
OS421840-NS	UItOS - 3.5MM NON-LOCKING SCREW 40MM - IN TRAY	8	Non-Sterile
OS421845-NS	UItOS - 3.5MM NON-LOCKING SCREW 45MM - IN TRAY	8	Non-Sterile
OS421850-NS	UItOS - 3.5MM NON-LOCKING SCREW 50MM - IN TRAY	8	Non-Sterile
OS421710	UItOS - 3.5MM LOCKING SCREW 10MM	2	Sterile
OS421712	UItOS - 3.5MM LOCKING SCREW 12MM	2	Sterile

OS PART NO.	PART DESCRIPTION	QTY./PACK	STERILITY
OS421714	UItOS - 3.5MM LOCKING SCREW 14MM	2	Sterile
OS421716	UItOS - 3.5MM LOCKING SCREW 16MM	2	Sterile
OS421718	UItOS - 3.5MM LOCKING SCREW 10MM	2	Sterile
OS421720	UItOS - 3.5MM LOCKING SCREW 20MM	2	Sterile
OS421722	UItOS - 3.5MM LOCKING SCREW 20MM	2	Sterile
OS421722	UItOS - 3.5MM LOCKING SCREW 22MM	2	Sterile
OS421724	UItOS - 3.5MM LOCKING SCREW 24MM	2	Sterile
OS421726	UItOS - 3.5MM LOCKING SCREW 26MM	2	Sterile
OS421728	UItOS - 3.5MM LOCKING SCREW 28MM	2	Sterile
OS421730	UItOS - 3.5MM LOCKING SCREW 30MM	2	Sterile
OS421735	UItOS - 3.5MM LOCKING SCREW 35MM	2	Sterile
OS421740	UItOS - 3.5MM LOCKING SCREW 40MM	2	Sterile
OS421745	UItOS - 3.5MM LOCKING SCREW 45MM	2	Sterile
OS421750	UItOS - 3.5MM LOCKING SCREW 50MM	2	Sterile
OS421710-NS	UItOS - 3.5MM LOCKING SCREW 10MM - IN TRAY	8	Non-Sterile
OS421712-NS	UItOS - 3.5MM LOCKING SCREW 12MM - IN TRAY	8	Non-Sterile
OS421714-NS	UItOS - 3.5MM LOCKING SCREW 14MM - IN TRAY	8	Non-Sterile
OS421716-NS	UItOS - 3.5MM LOCKING SCREW 16MM - IN TRAY	8	Non-Sterile
OS421718-NS	UItOS - 3.5MM LOCKING SCREW 10MM - IN TRAY	8	Non-Sterile
OS421720-NS	UItOS - 3.5MM LOCKING SCREW 20MM - IN TRAY	8	Non-Sterile
OS421722-NS	UItOS - 3.5MM LOCKING SCREW 20MM - IN TRAY	8	Non-Sterile
OS421722-NS	UItOS - 3.5MM LOCKING SCREW 22MM - IN TRAY	8	Non-Sterile
OS421724-NS	UItOS - 3.5MM LOCKING SCREW 24MM - IN TRAY	8	Non-Sterile
OS421726-NS	UItOS - 3.5MM LOCKING SCREW 26MM - IN TRAY	8	Non-Sterile
OS421728-NS	UItOS - 3.5MM LOCKING SCREW 28MM - IN TRAY	8	Non-Sterile
OS421730-NS	UItOS - 3.5MM LOCKING SCREW 30MM - IN TRAY	8	Non-Sterile
OS421735-NS	UItOS - 3.5MM LOCKING SCREW 35MM - IN TRAY	8	Non-Sterile
OS421740-NS	UItOS - 3.5MM LOCKING SCREW 40MM - IN TRAY	8	Non-Sterile
OS421745-NS	UItOS - 3.5MM LOCKING SCREW 45MM - IN TRAY	8	Non-Sterile
OS421750-NS	UItOS - 3.5MM LOCKING SCREW 50MM - IN TRAY	8	Non-Sterile

OS PART NO.	PART DESCRIPTION	QTY./PACK	STERILITY
OS326925	UItOS - 3.5MM CANNULATED SCREW 25MM SELF DRILL/TAP	2	Sterile
OS326928	UItOS - 3.5MM CANNULATED SCREW 28MM SELF DRILL/TAP	2	Sterile
OS326930	UItOS - 3.5MM CANNULATED SCREW 30MM SELF DRILL/TAP	2	Sterile
OS326932	UItOS - 3.5MM CANNULATED SCREW 32MM SELF DRILL/TAP	2	Sterile
OS326934	UItOS - 3.5MM CANNULATED SCREW 34MM SELF DRILL/TAP	2	Sterile
OS326936	UItOS - 3.5MM CANNULATED SCREW 36MM SELF DRILL/TAP	2	Sterile
OS326938	UItOS - 3.5MM CANNULATED SCREW 38MM SELF DRILL/TAP	2	Sterile
OS326940	UItOS - 3.5MM CANNULATED SCREW 40MM SELF DRILL/TAP	2	Sterile
OS326945	UItOS - 3.5MM CANNULATED SCREW 45MM SELF DRILL/TAP	2	Sterile
OS326950	UItOS - 3.5MM CANNULATED SCREW 50MM SELF DRILL/TAP	2	Sterile
OS326955	UItOS - 3.5MM CANNULATED SCREW 55MM SELF DRILL/TAP	2	Sterile
OS326960	UItOS - 3.5MM CANNULATED SCREW 60MM SELF DRILL/TAP	2	Sterile
OS326925-NS	UItOS - 3.5MM CANNULATED SCREW 25MM SELF DRILL/TAP - IN TRAY	8	Non-Sterile
OS326928-NS	UItOS - 3.5MM CANNULATED SCREW 28MM SELF DRILL/TAP - IN TRAY	8	Non-Sterile
OS326930-NS	UItOS - 3.5MM CANNULATED SCREW 30MM SELF DRILL/TAP - IN TRAY	8	Non-Sterile
OS326932-NS	UItOS - 3.5MM CANNULATED SCREW 32MM SELF DRILL/TAP - IN TRAY	8	Non-Sterile
OS326934-NS	UItOS - 3.5MM CANNULATED SCREW 34MM SELF DRILL/TAP- IN TRAY	8	Non-Sterile
OS326936-NS	UItOS - 3.5MM CANNULATED SCREW 36MM SELF DRILL/TAP- IN TRAY	8	Non-Sterile
OS326938-NS	UItOS - 3.5MM CANNULATED SCREW 38MM SELF DRILL/TAP- IN TRAY	8	Non-Sterile
OS326940-NS	UItOS - 3.5MM CANNULATED SCREW 40MM SELF DRILL/TAP- IN TRAY	8	Non-Sterile
OS326945-NS	UItOS - 3.5MM CANNULATED SCREW 45MM SELF DRILL/TAP- IN TRAY	8	Non-Sterile
OS326950-NS	UItOS - 3.5MM CANNULATED SCREW 50MM SELF DRILL/TAP - IN TRAY	8	Non-Sterile
OS326955-NS	UItOS - 3.5MM CANNULATED SCREW 55MM SELF DRILL/TAP - IN TRAY	8	Non-Sterile
OS326960-NS	UItOS - 3.5MM CANNULATED SCREW 60MM SELF DRILL/TAP - IN TRAY	8	Non-Sterile

OS PART NO.	PART DESCRIPTION	QTY./PACK	STERILITY
OS421300	UItOS - ARTHRODESIS WEDGE PLATE - 0MM - AWP	1	Sterile
OS421302	UItOS - ARTHRODESIS WEDGE PLATE - 2MM - AWP	1	Sterile
OS421304	UItOS - ARTHRODESIS WEDGE PLATE - 4MM - AWP	1	Sterile
OS421306	UItOS - ARTHRODESIS WEDGE PLATE - 6MM - AWP	1	Sterile
OS421308	UItOS - ARTHRODESIS WEDGE PLATE - 8MM - AWP	1	Sterile
OS421300-NS	UItOS - ARTHRODESIS WEDGE PLATE - 0MM - AWP - IN TRAY	1	Non-Sterile
OS421302-NS	UItOS - ARTHRODESIS WEDGE PLATE - 2MM - AWP - IN TRAY	1	Non-Sterile
OS421304-NS	UItOS - ARTHRODESIS WEDGE PLATE - 4MM - AWP - IN TRAY	1	Non-Sterile
OS421306-NS	UItOS - ARTHRODESIS WEDGE PLATE - 6MM - AWP - IN TRAY	1	Non-Sterile
OS421308-NS	UItOS - ARTHRODESIS WEDGE PLATE - 8MM - AWP - IN TRAY	1	Non-Sterile
OS422702	UItOS - GENERAL FUSION STRAIGHT PLATE - 2 HOLE - GFS	1	Sterile
OS422703	UItOS - GENERAL FUSION STRAIGHT PLATE - 4 HOLE - GFS	1	Sterile
OS422702-NS	UItOS - GENERAL FUSION STRAIGHT PLATE - 2 HOLE - GFS - IN TRAY	1	Non-Sterile
OS422703-NS	UItOS - GENERAL FUSION STRAIGHT PLATE - 4 HOLE - GFS - IN TRAY	1	Non-Sterile
OS422712	UItOS - GENERAL FUSION 'T' PLATE - 2 HOLE - GFT	1	Sterile
OS422714	UItOS - GENERAL FUSION 'T' PLATE - 4 HOLE - GFT	1	Sterile
OS422716	UItOS - GENERAL FUSION 'T' PLATE - 6 HOLE - GFT	1	Sterile
OS422712-NS	UItOS - GENERAL FUSION 'T' PLATE - 2 HOLE - GFT - IN TRAY	1	Non-Sterile
OS422714-NS	UItOS - GENERAL FUSION 'T' PLATE - 4 HOLE - GFT - IN TRAY	1	Non-Sterile
OS422716-NS	UItOS - GENERAL FUSION 'T' PLATE - 6 HOLE - GFT - IN TRAY	1	Non-Sterile

OS PART NO.	PART DESCRIPTION	QTY./PACK	STERILITY
OS42260SL	UItOS - METATARSOPHALANGEAL PLATE - SMALL LEFT - MTP	1	Sterile
OS42260SR	UItOS - METATARSOPHALANGEAL PLATE - SMALL RIGHT - MTP	1	Sterile
OS42260ML	UItOS - METATARSOPHALANGEAL PLATE - MEDIUM LEFT - MTP	1	Sterile
OS42260MR	UItOS - METATARSOPHALANGEAL PLATE - MEDIUM RIGHT - MTP	1	Sterile
OS42260LL	UItOS - METATARSOPHALANGEAL PLATE - LARGE LEFT - MTP	1	Sterile
OS42260LR	UItOS - METATARSOPHALANGEAL PLATE - LARGE RIGHT - MTP	1	Sterile
OS42260SL-NS	UItOS - METATARSOPHALANGEAL PLATE - SMALL LEFT - MTP - IN TRAY	1	Non-Sterile
OS42260SR-NS	UItOS - METATARSOPHALANGEAL PLATE - SMALL RIGHT - MTP - IN TRAY	1	Non-Sterile
OS42260ML-NS	UItOS - METATARSOPHALANGEAL PLATE - MEDIUM LEFT - MTP - IN TRAY	1	Non-Sterile
OS42260MR-NS	UItOS - METATARSOPHALANGEAL PLATE - MEDIUM RIGHT - MTP - IN TRAY	1	Non-Sterile
OS42260LL-NS	UItOS - METATARSOPHALANGEAL PLATE - LARGE LEFT - MTP - IN TRAY	1	Non-Sterile
OS42260LR-NS	UItOS - METATARSOPHALANGEAL PLATE - LARGE RIGHT - MTP - IN TRAY	1	Non-Sterile
OS422400	UItOS - OPENING WEDGE LOCKING PLATE - 0MM - OWL	1	Sterile
OS422403	UItOS - OPENING WEDGE LOCKING PLATE - 3MM - OWL	1	Sterile
OS422404	UItOS - OPENING WEDGE LOCKING PLATE - 4MM - OWL	1	Sterile
OS422405	UItOS - OPENING WEDGE LOCKING PLATE - 5MM - OWL	1	Sterile
OS422406	UItOS - OPENING WEDGE LOCKING PLATE - 6MM - OWL	1	Sterile
OS422407	UItOS - OPENING WEDGE LOCKING PLATE - 7MM - OWL	1	Sterile
OS422400-NS	UItOS - OPENING WEDGE LOCKING PLATE - 0MM - OWL - IN TRAY	1	Non-Sterile
OS422403-NS	UItOS - OPENING WEDGE LOCKING PLATE - 3MM - OWL - IN TRAY	1	Non-Sterile
OS422404-NS	UItOS - OPENING WEDGE LOCKING PLATE - 4MM - OWL - IN TRAY	1	Non-Sterile
OS422405-NS	UItOS - OPENING WEDGE LOCKING PLATE - 5MM - OWL - IN TRAY	1	Non-Sterile
OS422406-NS	UItOS - OPENING WEDGE LOCKING PLATE - 6MM - OWL - IN TRAY	1	Non-Sterile

OS PART NO.	PART DESCRIPTION	QTY./PACK	STERILITY
OS422407-NS	UItOS - OPENING WEDGE LOCKING PLATE - 7MM - OWL - IN TRAY	1	Non-Sterile
OS422800	UItOS - OPENING WEDGE PLATE - 0MM - OWP	1	Sterile
OS422825	UItOS - OPENING WEDGE PLATE - 2.5MM - OWP	1	Sterile
OS422803	UItOS - OPENING WEDGE PLATE - 3MM - OWP	1	Sterile
OS422804	UItOS - OPENING WEDGE PLATE - 4MM - OWP	1	Sterile
OS422805	UItOS - OPENING WEDGE PLATE - 5MM - OWP	1	Sterile
OS422807	UItOS - OPENING WEDGE PLATE - 7MM - OWP	1	Sterile
OS422800-NS	UItOS - OPENING WEDGE PLATE - 0MM - OWP - IN TRAY	1	Non-Sterile
OS422825-NS	UItOS - OPENING WEDGE PLATE - 2.5MM - OWP - IN TRAY	1	Non-Sterile
OS422803-NS	UItOS - OPENING WEDGE PLATE - 3MM - OWP - IN TRAY	1	Non-Sterile
OS422804-NS	UItOS - OPENING WEDGE PLATE - 4MM - OWP - IN TRAY	1	Non-Sterile
OS422805-NS	UItOS - OPENING WEDGE PLATE - 5MM - OWP - IN TRAY	1	Non-Sterile
OS422807-NS	UItOS - OPENING WEDGE PLATE - 7MM - OWP - IN TRAY	1	Non-Sterile
OS421608	UItOS - CALCANEAL STEP PLATE - 8MM - CSP	1	Sterile
OS421610	UItOS - CALCANEAL STEP PLATE - 10MM - CSP	1	Sterile
OS421612	UItOS - CALCANEAL STEP PLATE - 12MM - CSP	1	Sterile
OS421608-NS	UItOS - CALCANEAL STEP PLATE - 8MM - CSP - IN TRAY	1	Non-Sterile
OS421610-NS	UItOS - CALCANEAL STEP PLATE - 10MM - CSP - IN TRAY	1	Non-Sterile
OS421612-NS	UItOS - CALCANEAL STEP PLATE - 12MM - CSP - IN TRAY	1	Non-Sterile
OS42150XS	UItOS - GENERAL FUSION 'X' PLATE - EXTRA SMALL - GFX	1	Sterile
OS42150S	UItOS - GENERAL FUSION 'X' PLATE - SMALL - GFX	1	Sterile
OS42150M	UItOS - GENERAL FUSION 'X' PLATE - MEDIUM - GFX	1	Sterile
OS42150L	UItOS - GENERAL FUSION 'X' PLATE - LARGE - GFX	1	Sterile
OS42150XS-NS	UItOS - GENERAL FUSION 'X' PLATE - EXTRA SMALL - GFX - IN TRAY	1	Non-Sterile
OS42150S-NS	UItOS - GENERAL FUSION 'X' PLATE - SMALL - GFX - IN TRAY	1	Non-Sterile
OS42150M-NS	UItOS - GENERAL FUSION 'X' PLATE - MEDIUM - GFX - IN TRAY	1	Non-Sterile
OS42150L-NS	UItOS - GENERAL FUSION 'X' PLATE - LARGE - GFX - IN TRAY	1	Non-Sterile

OS PART NO.	PART DESCRIPTION	QTY./PACK	STERILITY
OS421000L	UItOS - LAPIDIUS PLATE - OMM LEFT - LAP	1	Sterile
OS421000R	UItOS - LAPIDIUS PLATE - OMM RIGHT - LAP	1	Sterile
OS421002L	UItOS - LAPIDIUS PLATE - 2MM LEFT - LAP	1	Sterile
OS421002R	UItOS - LAPIDIUS PLATE - 2MM RIGHT - LAP	1	Sterile
OS421004L	UItOS - LAPIDIUS PLATE - 4MM LEFT - LAP	1	Sterile
OS421004R	UItOS - LAPIDIUS PLATE - 4MM RIGHT - LAP	1	Sterile
OS421006L	UItOS - LAPIDIUS PLATE - 6MM LEFT - LAP	1	Sterile
OS421006R	UItOS - LAPIDIUS PLATE - 6MM RIGHT - LAP	1	Sterile
OS421000L-NS	UItOS - LAPIDIUS PLATE - OMM LEFT - LAP - IN TRAY	1	Non-Sterile
OS421000R-NS	UItOS - LAPIDIUS PLATE - OMM RIGHT - LAP - IN TRAY	1	Non-Sterile
OS421002L-NS	UItOS - LAPIDIUS PLATE - 2MM LEFT - LAP - IN TRAY	1	Non-Sterile
OS421002R-NS	UItOS - LAPIDIUS PLATE - 2MM RIGHT - LAP - IN TRAY	1	Non-Sterile
OS421004L-NS	UItOS - LAPIDIUS PLATE - 4MM LEFT - LAP - IN TRAY	1	Non-Sterile
OS421004R-NS	UItOS - LAPIDIUS PLATE - 4MM RIGHT - LAP - IN TRAY	1	Non-Sterile
OS421006L-NS	UItOS - LAPIDIUS PLATE - 6MM LEFT - LAP - IN TRAY	1	Non-Sterile
OS421006R-NS	UItOS - LAPIDIUS PLATE - 6MM RIGHT - LAP - IN TRAY	1	Non-Sterile
OS421406	UItOS - REARFOOT RECONSTRUCTION PLATE - 6 HOLE - RRP	1	Sterile
OS421408	UItOS - REARFOOT RECONSTRUCTION PLATE - 8 HOLE - RRP	1	Sterile
OS421414	UItOS - REARFOOT RECONSTRUCTION PLATE - 14 HOLE - RRP	1	Sterile
OS421406-NS	UItOS - REARFOOT RECONSTRUCTION PLATE - 6 HOLE - RRP - IN TRAY	1	Non-Sterile
OS421408-NS	UItOS - REARFOOT RECONSTRUCTION PLATE - 8 HOLE - RRP - IN TRAY	1	Non-Sterile
OS421414-NS	UItOS - REARFOOT RECONSTRUCTION PLATE - 14 HOLE - RRP - IN TRAY	1	Non-Sterile
OS421512	UItOS - TARSAL FUSION PLATE - 12MM - TFP	1	Sterile
OS421514	UItOS - TARSAL FUSION PLATE - 14MM - TFP	1	Sterile
OS421516	UItOS - TARSAL FUSION PLATE - 16MM - TFP	1	Sterile

OS PART NO.	PART DESCRIPTION	QTY./PACK	STERILITY
OS421512-NS	UItOS - TARSAL FUSION PLATE - 12MM - TFP - IN TRAY	1	Non-Sterile
OS421514-NS	UItOS - TARSAL FUSION PLATE - 14MM - TFP - IN TRAY	1	Non-Sterile
OS421516-NS	UItOS - TARSAL FUSION PLATE - 16MM - TFP - IN TRAY	1	Non-Sterile
OS421112	UItOS - UNIVERSAL LOCKING PLATE - 12MM - ULP	1	Sterile
OS421116	UItOS - UNIVERSAL LOCKING PLATE - 16MM - ULP	1	Sterile
OS421120	UItOS - UNIVERSAL LOCKING PLATE - 20MM - ULP	1	Sterile
OS421124	UItOS - UNIVERSAL LOCKING PLATE - 24MM - ULP	1	Sterile
OS421130	UItOS - UNIVERSAL LOCKING PLATE - 30MM - ULP	1	Sterile
OS421112-NS	UItOS - UNIVERSAL LOCKING PLATE - 12MM - ULP - IN TRAY	1	Non-Sterile
OS421116-NS	UItOS - UNIVERSAL LOCKING PLATE - 16MM - ULP - IN TRAY	1	Non-Sterile
OS421120-NS	UItOS - UNIVERSAL LOCKING PLATE - 20MM - ULP - IN TRAY	1	Non-Sterile
OS421124-NS	UItOS - UNIVERSAL LOCKING PLATE - 24MM - ULP - IN TRAY	1	Non-Sterile
OS421130-NS	UItOS - UNIVERSAL LOCKING PLATE - 30MM - ULP - IN TRAY	1	Non-Sterile

OrthoSolutions
Group



Ortho Solutions Ltd - UK
Unit 5, West Station Business Park,
Spital Road, Essex. CM9 6FF

Tel: +44 (0)1621 843 599
Fax: +44 (0)1621 858 953
Email: sales@orthosol.com

Ortho Solutions Inc - USA
330 Franklin Turnpike
Mahwah, NJ 07430

Tel: +1 (201) - 258-7305
Fax: +1 (201) - 335-0759
Email: us.sales@orthosol.com

Ortho Solutions Pty - Australia
PO Box 1330, Milton,
QLD. 4064

Tel: +61 (0) 408846821
Email: salespty@orthosol.com

© 2013 Ortho Solutions Ltd, and the OS logo are
trademarks of Ortho Solutions Ltd.

www.orthosolutions.com

CE
0120



Rx only

Caution: Federal Law (USA) restricts this device
to sale by or on the order of a physician